



Installing Intelviewer on Desktop


1. Using link provided, download IntelViewer installer and **Run** once complete
2. Click **Next**

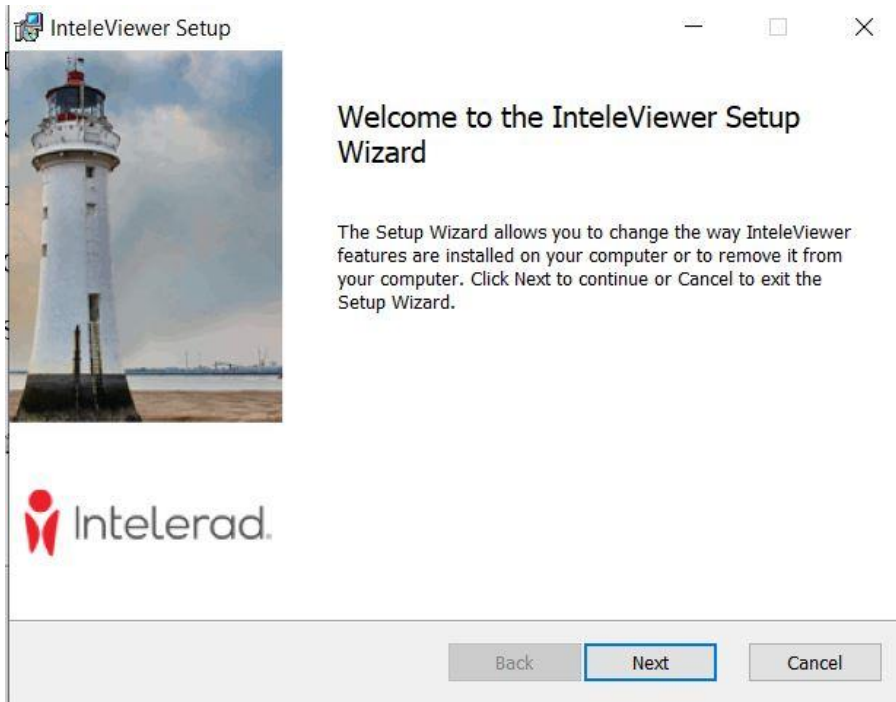


3. Tick the **I accept the terms in the License Agreement** checkbox and click **Next**

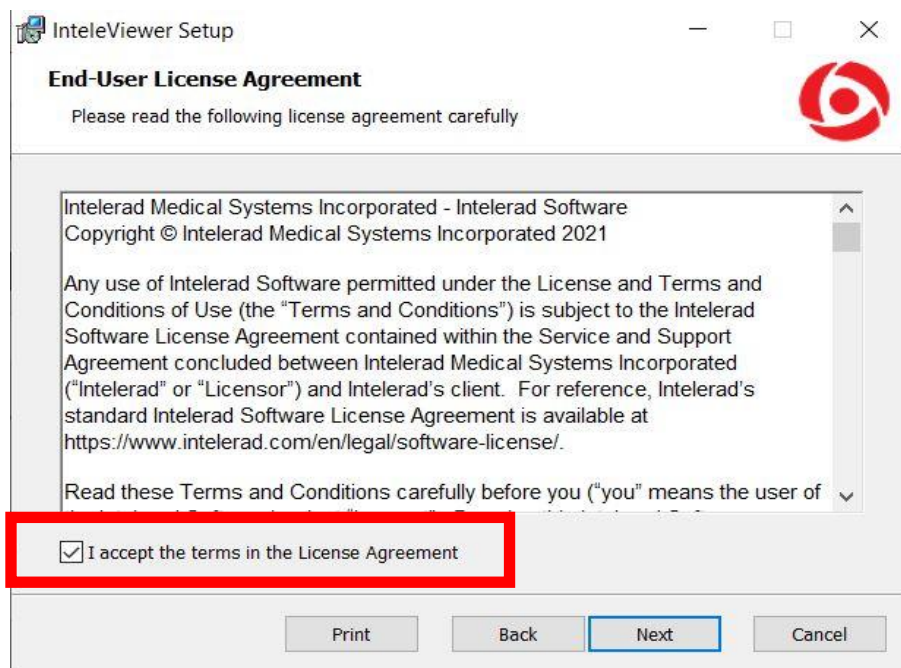


4. Select the desired destination folder for setup and click **Next**

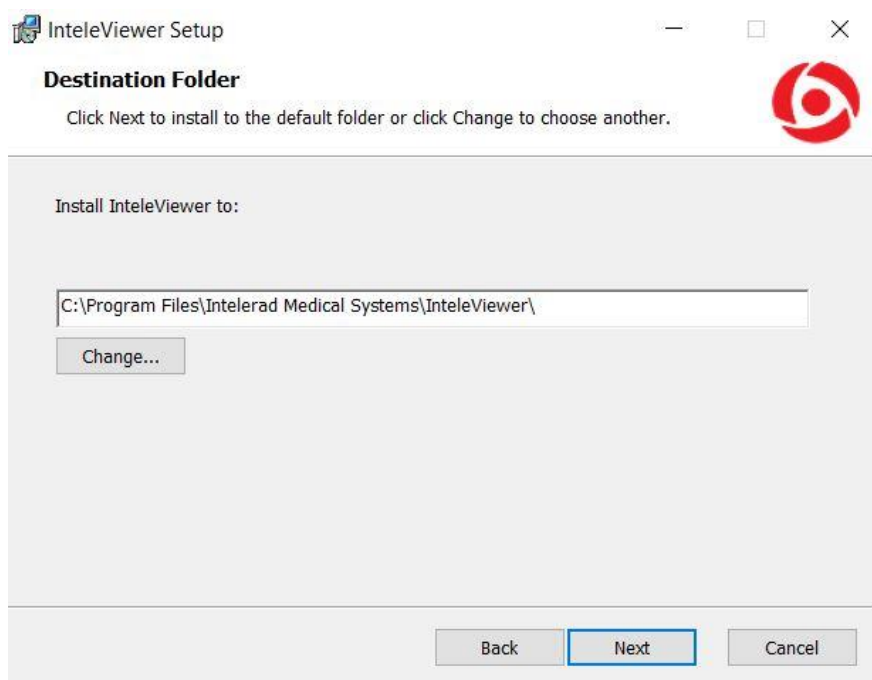


5. Select **All Users** and click **Next**

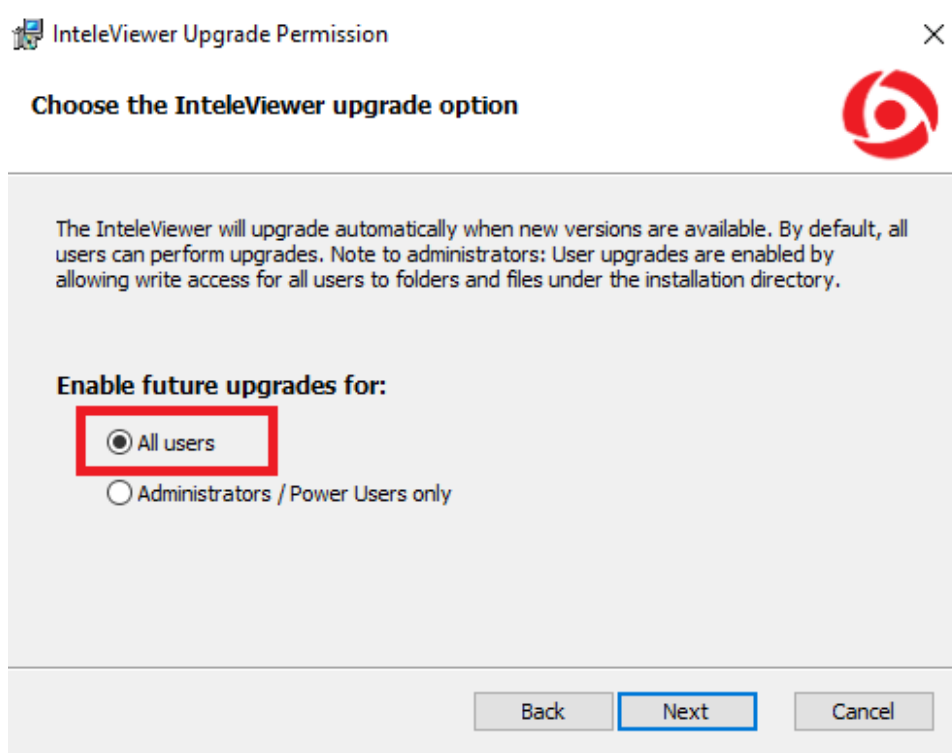


6. Select **Enable DICOM Service on port 5035** and click **Next**

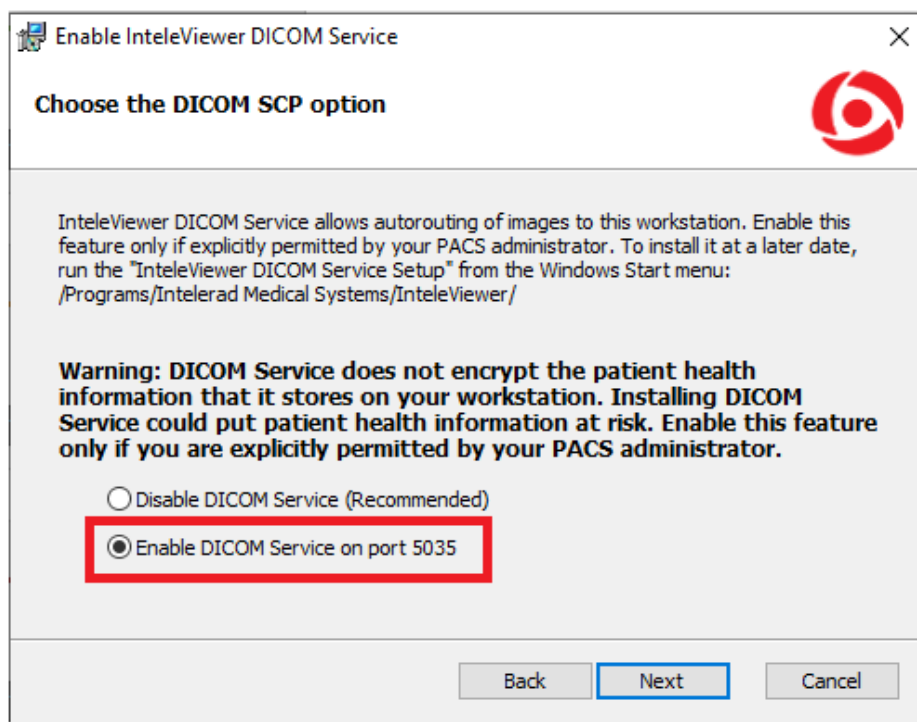


7. Select **Create shortcuts on my desktop** and click **Next**

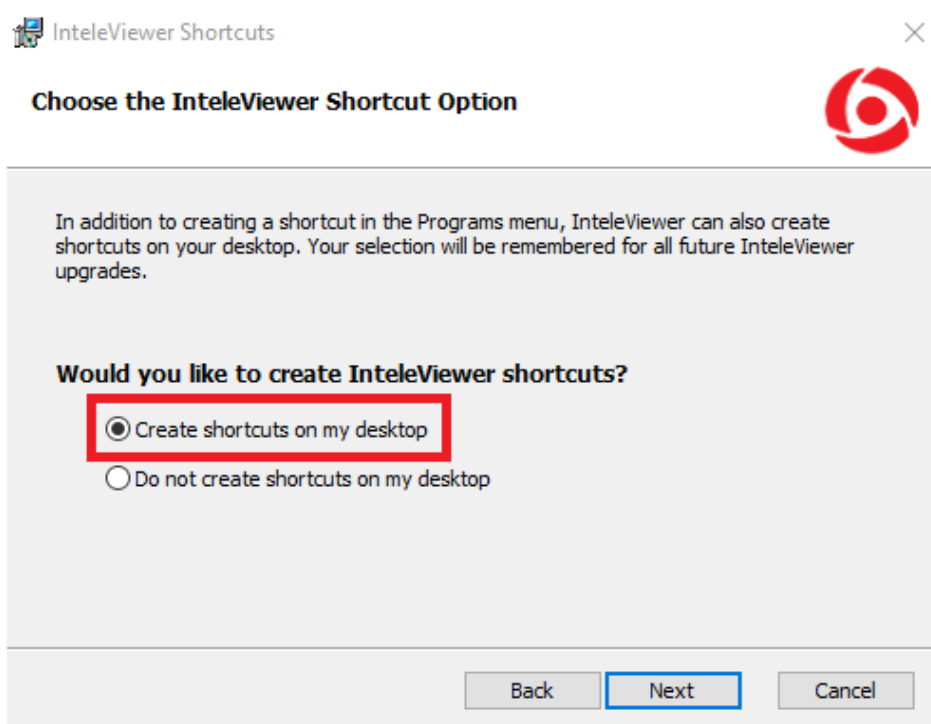

8. Click **Install**

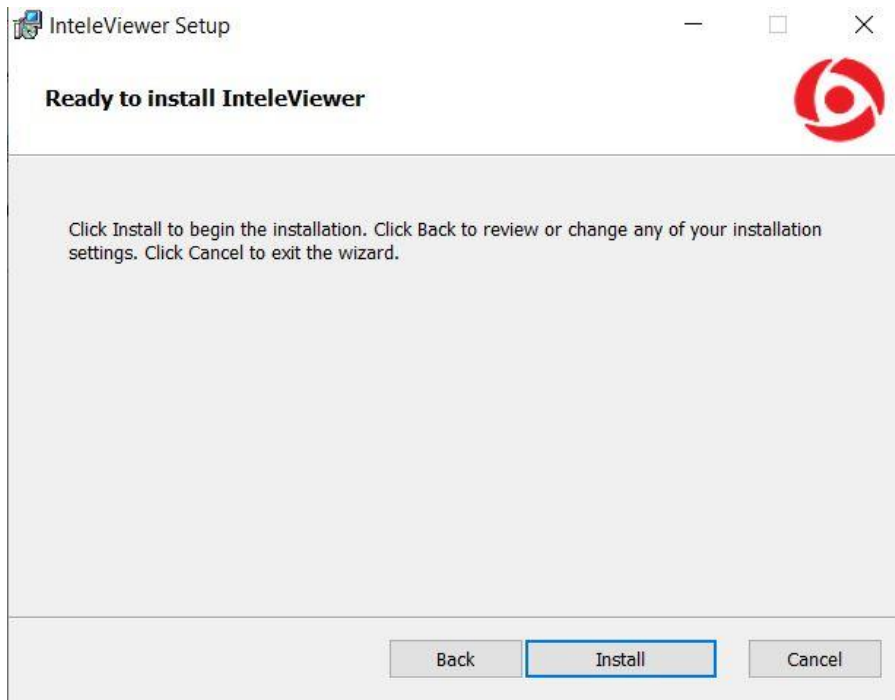


9. Click **Finish**

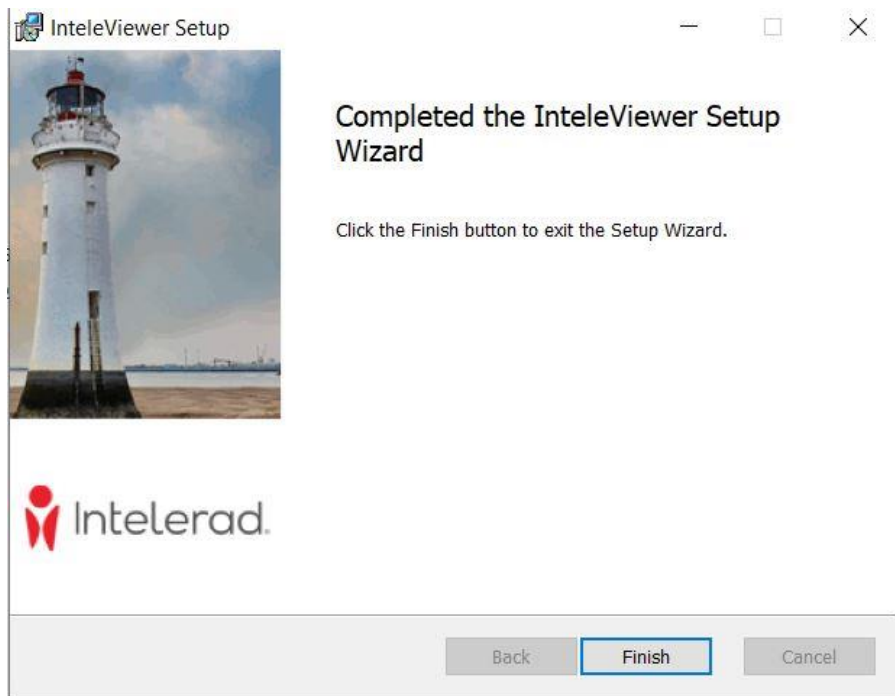

10. IntelViewer icon should now appear on the desktop

NOTE: intelViewer will prompt you to make a bookmark following install


- Click ***Enter a New Server***

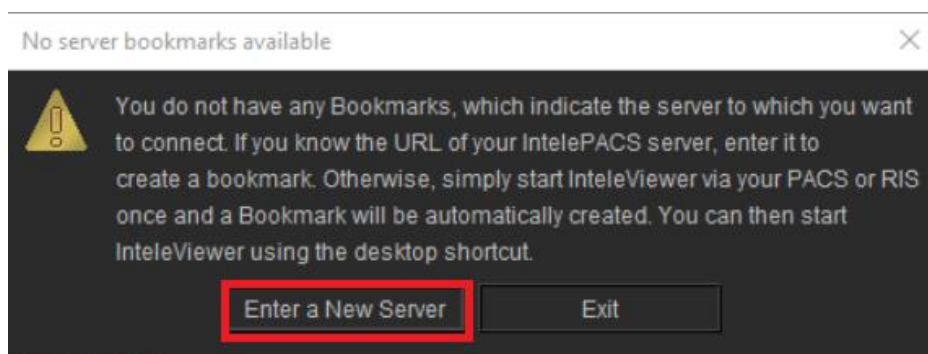


Adding Synergy Radiology Bookmark on IntelViewer

Instructions to add a bookmark:

1. Launch IntelViewer and click ***Edit Bookmarks*** from the menu on the left

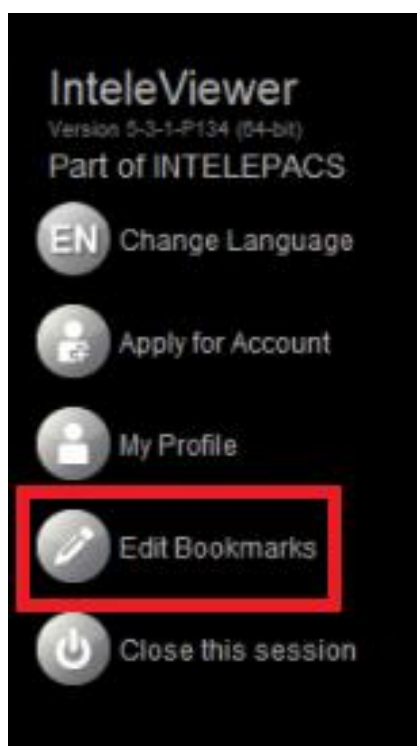


2. Click **Add**

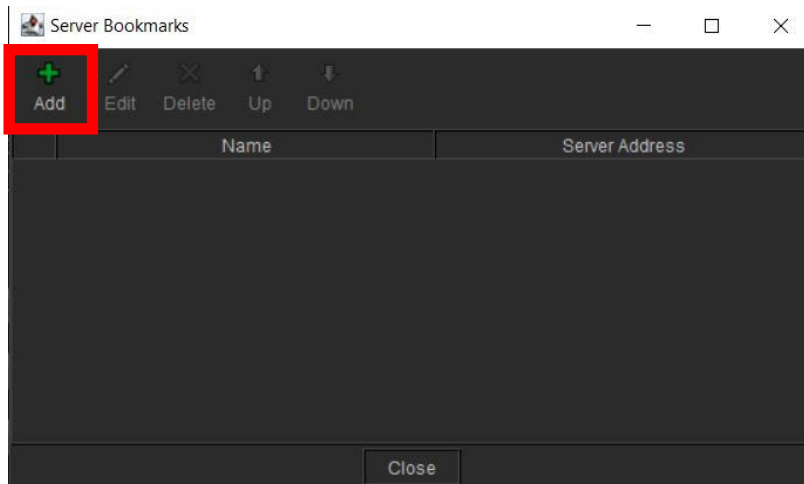

3. Enter in the below details:

- a. **Server URL:** `https://pacs.synrad.com.au`
- b. Tick **Protect Bookmark**
- c. Tick **Use Custom Server Name**
- d. **Name:** Synergy Radiology
- e. Click **Ok**

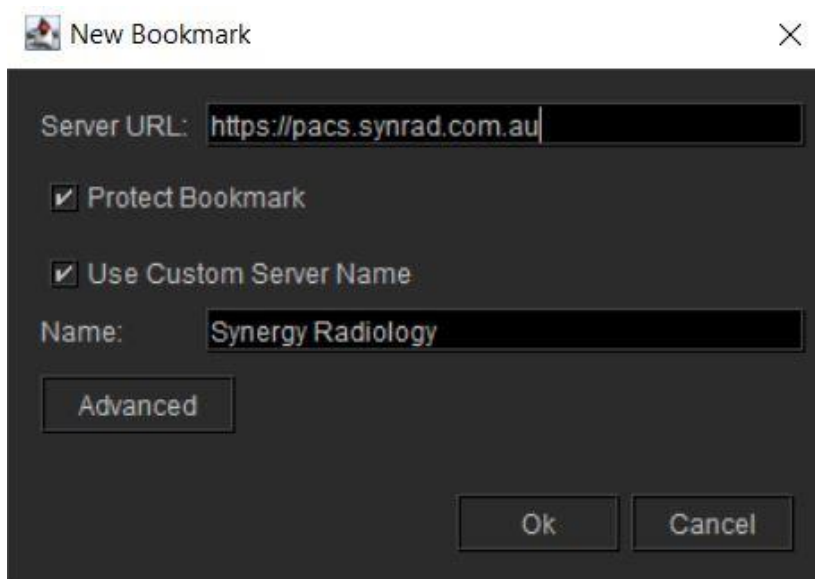

4. Bookmarks will now appear on the right of the interface. Use this to login into IntelViewer

