



Adding Synergy Radiology Bookmark on IntelViewer

Instructions to add a bookmark:


1. Launch IntelViewer and click **Edit Bookmarks** from the menu on the left

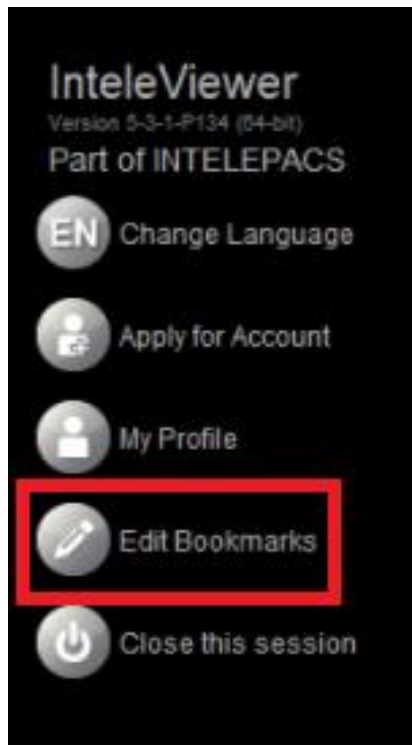

2. Click **Add**

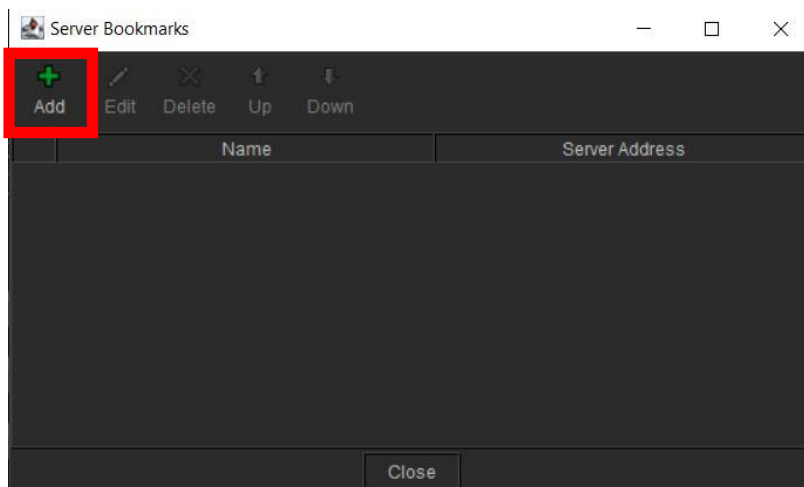

3. Enter in the below details:
 - a. **Server URL:** https://pacs.synrad.com.au
 - b. Tick **Protect Bookmark**
 - c. Tick **Use Custom Server Name**
 - d. **Name:** Synergy Radiology
 - e. Click **Ok**

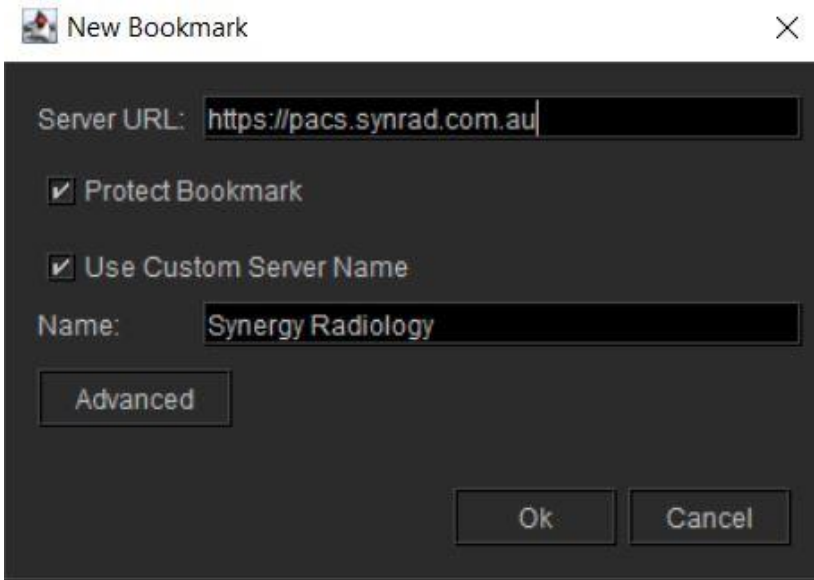

4. Bookmarks will now appear on the right of the interface. Use this to login into IntelViewer

